

## Introduction

The Days Healthcare 512 economy mobile commodes are two piece commodes supplied with 4 × 12 cm (5") front and rear brake castors, detachable armrests with budget style over-seat. hook-on footrests are available as accessories (ACC/10).

## Operation

- ☛ **WARNING!** - *This product should always be used under supervision.*

### Brakes/Castors

The 512DBA mobile commodes are fitted with fully braked castors. Each castor can be braked by pushing down the rear lever on the castor. This prevents the wheels from rotating or swivelling. To release the brake the locking lever should be pushed downwards.

### Entering or Leaving the Seat

- ☛ **WARNING!** – *Brakes should always be applied when entering or leaving the chair.*

If footrests are fitted they should be folded up or removed to allow easier entry or exit.

The armrest can be removed to allow side transfer of the user.

### Bowl Removal

The commode bowl is supplied with a lid and can be removed by sliding out from the back of the seat.

### Armrests

The commode has two armrests, which slot into tubes by the side of the frame. The arms can be removed by pushing down the front spring lock then pulling them out of the tubes.

- ☛ **WARNING!** – *Ensure the arms are securely fitted into the tubes and the locks engaged before using the chair.*

### Footrests

Optional hook-on footrests (ACC/10) can be fitted on the chair.

Height adjustment of the footrests is possible by slackening the black hand knob then sliding the bottom section up or down and re-tightening the hand knob. For access the footrest plates can be swivelled up vertically.

## Maintenance

The Days Healthcare 512 economy mobile commode is designed for minimum maintenance and easy, modular replacement of parts for long life. It is manufactured from chrome plated steel.

To clean, wipe over with a soft cloth. Use general purpose neutral detergent and warm water.

Do not use harsh abrasives.

To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry.

Daily, before use, check the effective operation of the brakes and castors.

## Specification

Width: 53 cm (21"); Length: 58 cm (23"); Height: 91 cm (36"); Seat Height: 51 cm (20")  
Max. User Weight: 125 kg (19½ st)

## Warranty Information

All Days Healthcare commodes are warranted for one year from the date of purchase against faulty workmanship or materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.

