

## Model Types

51318 18" wide chrome plated frame, plastic commode seat with square bowl, upholstered over-seat and backrest, 130 kg (20 1/2 stone) maximum user weight.



## Operation

### Fitting of Detachable Backrest

The backrest is fitted by inserting the back into the frame and tightening up the screw with the hand knob, ensuring that the screw head is recessed in the hexagonal shape on the inside of the tube and the hand knob and washer is fitted on the outside.




845HB001 Hand Knob

☛ **WARNING!** - *These products should always be used under supervision.*

**Bowl Removal** - The commode bowl is removed by sliding out from the back of the seat.

### Armrests

The armrests slot into brackets by the side of the chair. The arms can be removed by pressing the lever towards the arm, raising the front side of the arm and then releasing the arm at the rear by pressing on the spring button. When re-fitting the arm, ensure that the spring button is locked in place. To lower the armrests only, press the lever towards the arm, raise the arm and then lower to one side until it is resting on the support bar.

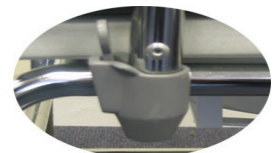

845PL002  
Armrest Release Clip

☛ **WARNING!**

– Ensure the arms are securely slotted into the brackets before use.


Do not attempt to lift the commode by the arms.

### Perfection Pan & Pan Rack

All commodes can be fitted with Perfection Pan Rack (Days Healthcare part no. ACC/2).

To fit the pan rack slide it onto the runners under the seat (SA154) and hook the rear crossbar over the plastic clip at the back of the commode seat.

The rack will take the Perfection Pan (Days Healthcare part no. 640) and the disposable pan.

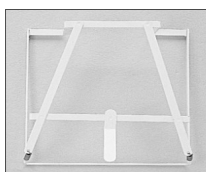

ACC/2



SA154



No. 640

## **Maintenance**

The Days Healthcare 51318 Static Commode is designed for minimum maintenance and easy, modular replacement of parts for long life. It is manufactured from chrome plated steel.

To clean, wipe over with a soft cloth. Use general purpose neutral detergent and warm water.

Do not use harsh abrasives.

To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry. Do not use any decontamination process that involves immersion.

Daily, before use, check that the commode is stable and none of the legs are bent. Check that the commode arms are located correctly.

The chair is designed to allow damaged or broken parts to be changed easily. If any parts are damaged please contact the supplier of the chair for replacement parts and fitting instructions.

## **Warranty Information**

The Days Healthcare model no. 51318 commode is warranted for one year from date of purchase due to faulty workmanship and materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non observance of the instructions set out above.