

Introduction

The Days Healthcare 521AHD Bariatric Commode is manufactured from powder coated steel tubing and fitted with a high comfort profile seat, removable padded over seat and upholstered backrest. The commode is supplied with a plastic bowl complete with a lid and a handle. The arms can be detached for ease of side transfer. The commode come with adjustable height legs, two of which are fitted with wheels for easy of manoeuvring in tight spaces.



Usage

Before use make sure the commode is stable on a flat horizontal floor. The pot must be located in the recesses on the two support bars under the toilet seat.

To adjust the height grasp the horseshoe clip between the thumb and forefinger and pull it out. Slide the leg in or out to the required position. Push the clip back firmly into the hole. Repeat the same procedure for the other legs making sure all four legs are adjusted to the same height.

The arms can be removed by taking out the locking clip then lifting them out of the sockets, enabling side transfer to/from the commode. Make sure the arms are securely refitted before putting any weight on them.

Maintenance

The Days Healthcare 521AHD Bariatric Commode is designed for minimum maintenance.

To clean, wipe over with a soft cloth. Use general purpose neutral detergent and warm water.

Do not use harsh abrasives.

To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry.

Do not store in sub-zero temperature. Periodically check for elongated or stressed adjustment holes. Check for split, worn or loose ferrules and loose or damaged horseshoe clip.

Specification

Overall Width: 76 cm;	Overall Length: 56 cm	Between Arms: 66 cm
Overall Height: 98.5 cm;	Seat Height: 45 - 58 cm	Seat to Top of Arms: 20 cm
Weight: 14.5 kg	Max. User Weight: 250kg	Seat to Top of Back: 40.5 cm

Warranty Information

The Days Healthcare 521AHD bariatric commode is warranted for one year from the date of purchase against faulty workmanship or materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.

