

Introduction

The Days Healthcare Perching Stools are manufactured from powder coated steel tubing and fitted with fully sealed seat and back. The seat is designed to be angled forward enabling the user to rise with ease.

Usage

To adjust the height grasp the horseshoe clip between the thumb and forefinger and pull it out. Slide the leg in or out to the required position. Push the clip back firmly into the hole. Repeat the same procedure for the other legs making sure all four legs are adjusted to the same height.



Safety Precautions

Ensure the stool is stable on a flat surface before transferring your weight on to the seat.

Do not overstretch sideways or backwards from sitting position as you could overbalance.

Do not attempt to move the stool whilst seated as the rubber ferrules grip the floor and could cause overbalance.

Maintenance

The Days Healthcare Perching Stools are designed for minimum maintenance.

To clean, wipe over with a soft cloth. Use general purpose neutral detergent and warm water. Do not use harsh abrasive materials.

To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry.

Periodically check for elongated or stressed adjustment holes. Check for split, worn or loose ferrules and loose or damaged horseshoe clips or bent legs that will make the stool unstable.

Specification

Model No.	Seat Height at Front	Width Between Arms	Max. User Weight
547A	56 - 70 cm	N/A	125 kg
548, 549, 549B	56 - 70 cm	40.5 cm	125 kg

Warranty Information

All Days Healthcare perching stools are warranted for one year from the date of purchase against faulty workmanship or materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.

