


## Model No. 587 Bath Board

### Introduction

The Days Healthcare 587 bath board is an adjustable width seat manufactured from non corrosive materials. It is capable of being adjusted to suit majority of bath widths by setting the individual posts into the required position to cater for the bath position whether free standing or located adjacent to a wall.

Please seek medical advice on the clinical and product suitability should you have any doubts.

### Directions of use

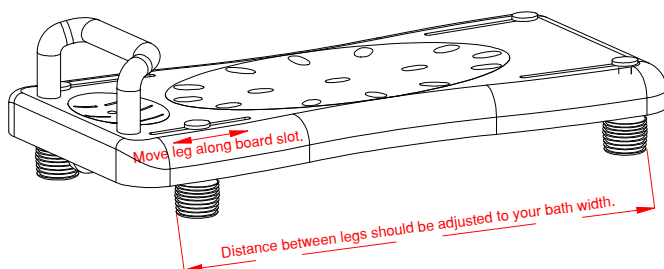

**WARNING:** Please follow assembly instructions on reverse side before use.

Locate and secure the individual posts into the required positions to suit your bath width and location. Please ensure there is no sideways movement of the seat and that the post surface is in contact with the bath walls. Both ends of the seat must be fully supported by the bath sides.

### Maintenance

Due to its simple design the only inspection required adjustment posts, to ensure they are correctly secured and are not split or deformed. Do not use if seat surface becomes cracked or damaged.

### Cleaning

To clean, wipe over with a soft cloth using warm soapy water. Do not use abrasive cleaners or those which are bleach based to avoid surface damage or discolouration.

### Warranty Information

The Days Healthcare model no 587 bath board is warranted for one year from date of purchase against faulty workmanship or materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.

Maximum User Weight: 190 kg

Available spare parts: None


## ASSEMBLY INSTRUCTIONS FOR 587 BATH BOARD

The bath board is supplied in flat back form so the adjustable legs and handle should be fixed to the board before use. No special tools are required. The following items are packed with the bath board.

1. Moulded plastic bath board x 1
2. Legs with rubber stoppers x 4
3. Flat faced plastic bolts x 4
4. Handle x 1
5. Plastic screws x 2

### Leg Assembly

The bolts are fitted through the slots at each corner of the board and the legs are then screwed into place from underneath the board as shown.

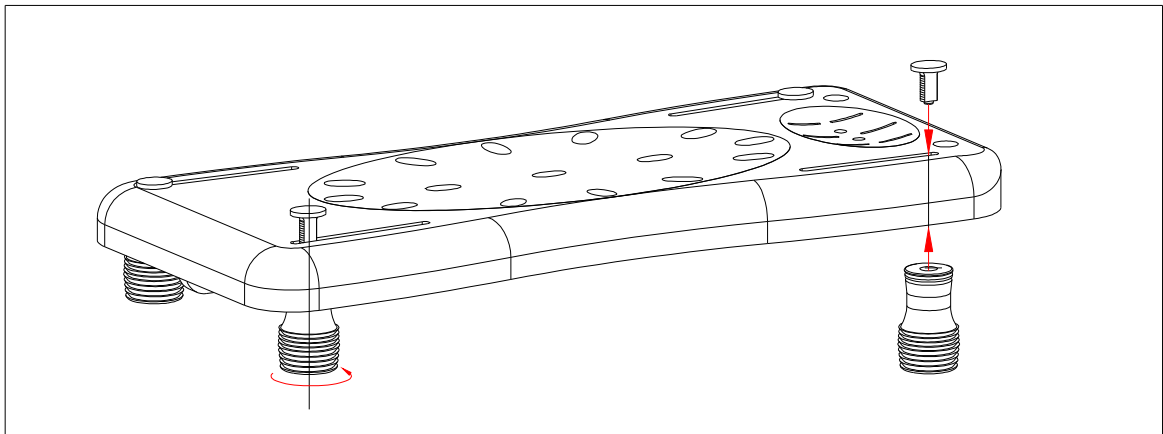

### Handle Assembly

- The handle is first fitted into the 2 locating holes at one end of the board as in fig. 2.
- The screws are fixed to handle from the underside of the board and can be tightened using a coin as in fig 3.

