



FAQs

Q. My bathlift fails to rise

A. Check that the airhose is connected to the Airflo compressor, check that the air hose is connected to the lever hand control and the "auto function" has been selected. (Battery version). Check that the airhose is not kinked or trapped.

Q. My bathlift fails to lower

A. Check that the Airflo batteries are sufficiently charged, check the airhose is not kinked or trapped.

Q. How can I prolong the life of my bath lift?

A. We recommend that you regularly clean the Archimedes Air after use. This can be done using any domestic bath cleaner. To assist the drying of the bath lift you should leave it fully inflated with the compressor switched off. **DO NOT** use hot air to dry.

Q. How do I get onto the Archimedes Air Bathlift?

A. When you transfer on and off the bathlift the side flaps should be horizontal with the seat and resting on the side of the bath.

Q. How can I make sure the suckers (suction cups) stick firmly to the bath?

A. We suggest you regularly wipe over the bottom of the four suckers (suction cups) on the base of the bath lift, as this will make sure they have the maximum adhesion onto your bath.

Q. I use a battery operated Airflo compressor with my bathlift. How can I ensure the battery does not go flat?

A. You should leave the Airflo compressor on charge when not in use. Once fully charged it will automatically cut off. If you have flattened the battery it should be put on charge immediately.

Q. Can bath oils and oil based products be used with my bathlift?

A. Yes, but we would recommend that you thoroughly clean the product after use.

Q. How many baths can I get out of 1 charge?

A. The amount of baths can vary on a number of factors; how much the user or users weight, how old the batteries are and most importantly, how well the batteries have been maintained and charged since purchase. We recommend that batteries are left on charge when not in use. Not only will this mean they are always ready to give enough power for several baths, it will also keep the batteries in optimum condition for a longer life.

Q. Can someone install my Archimedes for me?

A. The Archimedes is very simple to fit and most users will not require professional installation. You can fit the Archimedes in one piece, or easily dismantle it into 3 lighter parts to aid installation. Full fitting instructions are provided with the product in the User Manual. Fitting is as simple as lining the Archimedes up at the most appropriate position in the bath and pressing the suction cups down firmly to secure it.