

### Models

- ♦ 84618 18" seat width
  - ♦ 846F18 18" seat width, with footrests
  - ♦ 846DPR 18" seat width, with disposable panrack
  - ♦ 846FDPR 18" seat width, with disposable panrack and footrests
- (MAX USER WT - 160 Kg (25 st))**
- ♦ 84620 20" seat width
  - ♦ 846F20 20" seat width, with footrests
  - ♦ 84622 22" seat width
  - ♦ 846F22 22" seat width with footrests
- (MAX USER WT - 225 Kg (35 st))**

### Warranty Information

The Days Healthcare 846 range of Commode/Shower Chairs are warranted for one year from the date of purchase against faulty workmanship or material. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.

*Days Healthcare Ltd. is not responsible for any damage or injury due to improper or unsafe use of their Commodes.*



# ATLANTIC COMMUNE/SOWER CHAIR

MODEL 846 series



## INTRODUCTION


The Atlantic 846 range can be used for toileting away from the bathroom or used when wheeled over a standard toilet. Made from lightweight aluminium tubing fitted with plastic castors, aperture seat and plastic components makes it suitable for use in the shower.

Each model is supplied with a rear access plastic bowl, clip on backrest, aperture seat and detachable arms, which can be swung back or dropped down for easy access and side transfers.

## OPERATION


### Backrest

The backrest arrives detached from the frame. To fit the backrest to the frame, insert the vertical tube (fig 1) into the base frame (fig 2) and tighten up the hand knob (fig 3) until it is completely secure.

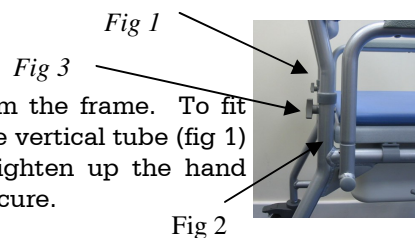

### Castors

Each model is fitted with four, fully braked castors. To prevent the wheels from rotating and swivelling, press down the lever on each castor. To release the castor, push the lever upwards.

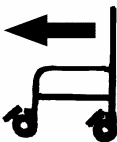

- ☛ **WARNING!** – Brakes should always be applied with the castors facing forward when entering or leaving the chair. With the brakes applied the commode may still slide on a smooth floor or if it is tipped forward by the user.

*Supervision is essential while getting on or off the commode.*

### Armrests

The multi-function design can be swung back, dropped down or removed for easy access to the seating area. The arm is locked in position by the front socket. To release, press the plastic lever to the rear and lift arm to remove from the socket. To remove the arm from the rear socket, depress the spring button at the rear and lift arm.



- ☛ **WARNING!** – Ensure the spring button and the front locking clip are fully locked in position and secure before use.

### Footrests

If fitted to your model.

Lift and swivel upwards until vertical to assist in gaining access in or out of the commode seat

- ☛ **WARNING!** - Do not stand on the footrest when transferring in or out of the chair.

### Bowl Removal

The commode bowl is located underneath the seat. Access is at the rear and it is removed or fitted by sliding in or out from the back of the seat.

### Disposable Pan Rack

A Disposable Pan Rack is fitted to the 18" seat model as an optional extra.

### Maintenance

The Days Healthcare 846 range of Commode/Shower Chairs are designed for minimum maintenance and easy, modular replacement of parts for long life.

Daily, before use, check the effective operation of the brakes and castors.

Ensure all clamps and fasteners are done up securely and that seat and backrest parts are clipped firmly in place.

If any parts are damaged please contact the supplier of the chair for replacement parts and fitting instructions.

### Cleaning

To clean, wipe over with a soft cloth. Use general purpose neutral detergent and warm water. The chair may be placed under running water at low pressure or showered down to assist cleaning.

Do not use harsh abrasives or chemicals.

To disinfect clean as above and wipe over with a neat solution of hypochlorite, rinse and dry.

- ♦ Both the backrest and commode seat can be unclipped for removal to assist in cleaning